

ThermalSoft[®] Gel Hot and Cold Packs Cleaning & Disinfecting Instructions

Cleaning: Use neutral soap and lukewarm water to clean ThermalSoft Gel packs. Rinse with water and allow to dry. Do not use harsh cleansers, solvents or detergents.

Disinfecting: Either phenolic or quaternary type disinfectants (In concentrations recommended by the manufacturer) may be used satisfactorily to disinfect the packs. ThermalSoft Gel packs may also be cleaned with a 1:10 dilution of household bleach containing 5.25% sodium hypochloride as recommended by the Center for Disease Control in Atlanta, Georgia. Do not soak the packs in the disinfectant longer than the manufacturer's recommendation. Thoroughly rinse the pack with warm water after disinfection. ***Disinfectants applied at full concentration or in highly concentrated solutions will decrease the useful life of the packs.***